

TS-DA-6-N Instructions for the biological monitoring of steam sterilization processes

Area of application:

The test system contains the biological indicator SIMICON ST/DA according to ISO 11138-3.

It is designed for the biological validation and routine monitoring of steam sterilization processes in small sterilizers type N.

Testing procedure:

1. Remove the card with the biological indicators from the envelope.
Don't open any of the single indicator pouches.
2. The indicator in the PE-bag serves as growth and transport control.
Do not sterilize this control indicator.
3. Place the card with the indicators in the sterilizer.
4. Start the sterilization process.
5. Fill in the test report.

Note: Be sure to enter all data concerning sterilizer model, temperature, pressure and time as well as the batch number of the test.

6. After the sterilization process is completed, remove the card with the biological indicators from the machine and send it with transport control and test report to SIMICON GMBH.

Note: Don't forget to add your address, telephone and fax number on the test report.

7. As soon as the biological assessment has been completed, SIMICON will return the test report including test results to you.