

Adapter set for cleaning and disinfection processes in Belimed / BHT washer disinfectors with pressure box system

Field of application:

Adapter set for use with PCD PM-RIBI-1 and PM-RIBI-2 in Belimed and BHT machines with pressure box system.

Instructions for use:

1. Unscrew the cap of the process challenge device (PCD).
2. Take the cleaning or biological indicator out of its pouch and insert it into the holding slot. Then screw the cap back onto the PCD tightly.
3. Fasten the stainless steel adapter to the PCD. Then screw it into the thread of the small PP-adapter.
4. Insert the small PP-adapter into the slot on the side of the pressure box so that the PCD can be placed in the load carrier as shown in figure 1.
5. Narrow the outlet of the pressure box into the Teflon tube by using the special mandrel as shown in figure 2. (The design of the mandrel may vary.)
6. Close the pressure box and make sure the adapter rests firmly in place.

Note: During the routine monitoring of Belimed / BHT washer disinfectors the second pressure box should be loaded with an endoscope.

7. Move the charging basket into the washer disinfectant and choose the **intensive** programme. **Programmes with a short cycle will not be sufficient for completely removing the test soil!**
8. Start the programme.
9. At the end of the programme take the indicator out of the PCD and assess the cleaning performance visually or send the biological indicator to a laboratory for assessment.



Figure 1:
Positioning of adapter and PCD in the WD



Figure 2:
Mandrel narrowing outlet of pressure box



Bild 3:
Adapter set