

Adapter for washer disinfectors for endoscopes with single channel adaptation

Field of application:

The adapter is for use with the process challenge devices PM-RIBI-1 and PM-RIBI-2 in washer disinfectors for endoscopes with single channel adaptation.

Instructions for use:

1. Attach the adapter to a representative cleaning nozzle of the charging basket.
2. Open the receptacle of the process challenge device.
3. Take the cleaning or biological indicator out of its pouch and insert it into the holding slot of the receptacle. Then screw the cap back onto the PCD tightly.
4. Connect the process challenge device to the adapter attached to the charging basket.
5. Move the charging basket into the WD and choose the **standard programme**.
Quick programmes like "ECO" are not suitable!
(During a routine testing of an Olympus ETD-WD at least 3 out of 6 cleaning nozzles should either be connected or blocked.)
6. Start the programme.
7. At the end of the programme, take the indicator out of the process challenge device and assess the cleaning performance visually or send the biological indicator to a laboratory for assessment.



Adapter



Adapter connected to process challenge device